

## ForeverPrep™

Item No. 345FP

*Quick & Easy Application of Prepzyme® Forever Wet for the Pretreatment of Soiled Instruments*

The Ruhof ForeverPrep™ Pretreatment Sprayer is an automated application system for Prepzyme® Forever Wet Humectant Pre-Cleaner which makes the job of coating surgical instruments fast and easy.

Prepzyme® Forever Wet is a neutral pH, multi-tiered enzymatic humectant spray which promotes the long lasting retention of moisture on soiled instruments and scopes thus helping to prevent the adhesion of bio-burden. This unique formulation gently coats instruments to maintain moisture making it an ideal pre-cleaner for soiled instruments during transport or when left for an extended period of time.

### FEATURES

- Applies Prepzyme® Forever Wet which keeps instruments moist for up to **72 HOURS**, helping to prevent the adhesion of bio-burden
- Quickly coats instrument trays in less than 8 seconds
- Optional spray gun for treating cannulated instruments such as suction tips, endoscopes, drill bits and rongeurs
- Eliminates manual pumping with trigger sprayers
- Ready to operate out of the box
- Safe 24 volt operation
- Can be wall mounted or placed on the counter; Convenient carry handle for mobile application
- System houses 1 gallon bottle of Forever Wet inside



Wide spray pattern to cover all instruments evenly







Optional cannulated instrument gun includes dosing tips for channel cleaning

## INSTRUCTIONS FOR USE (IFU)

1. Check chemical level in bottle.
2. Turn power on.
3. Hold applicator wand 6" – 8" over instruments and squeeze lever.
4. Coat instruments thoroughly, moving the applicator side to side.

### CAUTION:

Personal protective equipment should be worn when operating ForeverPrep™. Always point the ForeverPrep™ applicator wand away from you.

Dosing Tip		Cannulated Instrument Types
Straight 345PVCT		Forceps, rigid endoscopes, suction tips, trocars, tissue staplers
45 Degree 345PVCT45		Forceps, rigid endoscopes, Rongeurs,
Suction 345PVST		Suction tips
Rongeurs 345PVRA		Rongeur/Kerrison