


# InstruSponge™ ENDOZIME®

## An Enzymatic Sponge on a Flexible Wand for Endoscopes

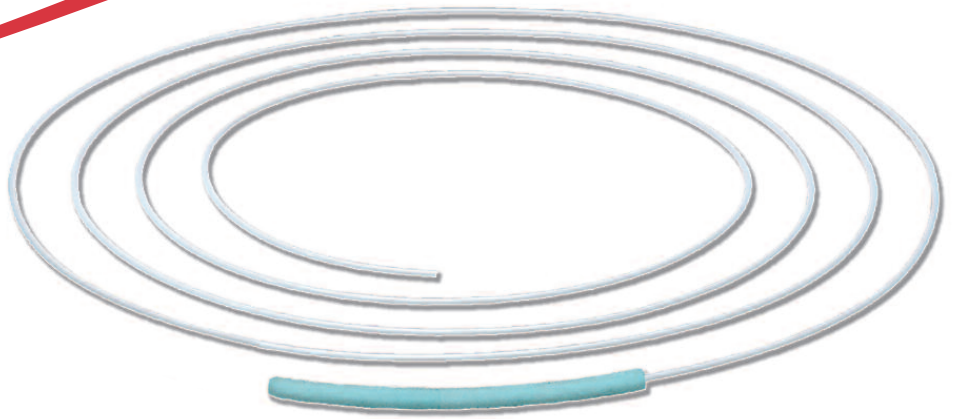

InstruSponge™ is impregnated with Endozime®AW Triple Plus® with APA and is attached to a flexible plastic wand that allows for easy manoeuvring through complex scope channels, to loosen and expel gross contaminants. Unlike conventional channel wire bristle brushes that can cause scarring on scope walls, InstruSponge™'s foam tip gently wipes clean the inside surface of the scope channel while instantly and actively digesting soil upon contact. Dipping the foam tip in water immediately activates the enzymes.

### CATALOGUE NO'S

**345ESS3F**  
3mm x 240cm

**345ESS4F**  
4mm x 240cm

**345ESS5F**  
5mm x 240cm



### DIRECTIONS FOR USE

- 1) Dip foam tip in water to activate enzymes.
- 2) Insert foam end into channel and run the length of channel until the sponge appears (see image).
- 3) After each passage, rinse foam tip to remove any visible debris before retracting and re-inserting it.
- 4) Continue passing through until there is no debris visible on foam tip.
- 5) Discard InstruSponge™ after each use.

**RUHOF**  
THE EXPERTS IN A BOTTLE

Peskett Solutions Ltd  
Unit 66 Hammonds Drive Ind. Est  
Eastbourne, East Sussex. BN23 6PW  
Tel: 01323 419918 Fax: 01323 439947  
E Mail: info@ruhof.co.uk Web: www.ruhof.co.uk