

InstruSponge™ ENDOZIME®

An Enzymatic Sponge on a Flexible Wand for Rigid and Take-apart Instruments



InstruSponge™ is impregnated with Endozime®AW Triple Plus® with APA and is attached to a flexible plastic wand that allows for easy manoeuvring through complex internal channels, cannulas and lumens to loosen and expel gross contaminants. Unlike conventional channel wire bristle brushes that can cause scarring on instrument walls, InstruSponge™'s foam tip gently wipes clean the inside surface of the take-apart channel while instantly and actively digesting soil upon contact. Dipping the foam tip in water immediately activates the enzymes.

CATALOGUE NO'S

345ESS1R	1 mm
345ESS015R	1.5 mm
345ESS0175R	1.75 mm
345ESS3R	3 mm
345ESS5R	5 mm
345ESS7R	7 mm
345ESS10R	10 mm
345ESS12R	12 mm
345ESS14R	14 mm



DIRECTIONS FOR USE

- 1) Dip foam tip in water to activate enzymes.
- 2) Insert foam end into channel and run length of channel.
- 3) Back and forth action will help dislodge any debris.
- 4) After each passage, rinse foam tip to remove any visible debris before retracting and re-inserting it.
- 5) Continue passing through until there is no debris visible on foam tip.
- 6) Discard InstruSponge™ after each use.

RUHOF
THE EXPERTS IN A BOTTLE

Peskett Solutions Ltd
Unit 66 Hammonds Drive Ind. Est
Eastbourne, East Sussex. BN23 6PW
Tel: 01323 419918 Fax: 01323 439947
E Mail: info@ruhof.co.uk Web: www.ruhof.co.uk