

ForeverPrep™

Quick & Easy Application of Prepclean and Prepzyme® Forever Wet for the Pretreatment of Soiled Instruments


The Ruhof ForeverPrep™ Pretreatment Sprayer is an automated application system for Forever Wet Humectant Pre-Cleaner which makes the job of coating surgical instruments fast and easy.

Prepclean Forever Wet is a neutral pH and Prepzyme Forever Wet is a neutral pH (multi-tiered enzymatic) humectant spray which promotes the long lasting retention of moisture on soiled instruments and scopes thus helping to prevent the adhesion of bio-burden. This unique formulation gently coats instruments to maintain moisture making it an ideal pre-cleaner for soiled instruments during transport or when left for an extended period of time.

FEATURES

- Applying Prepclean or Prepzyme Forever Wet will keep instruments moist for up to 48 hours unwrapped and 72 hours wrapped.
- Quickly coats instrument trays in less than 8 seconds.
- Optional spray gun for treating cannulated instruments such as suction tips, endoscopes, drill bits and rongeurs.
- Eliminates manual pumping with trigger sprayers.
- Ready to operate out of the box.
- Safe 24 volt operation.
 - Can be wall mounted or placed on the counter. Convenient carry handle for mobile application

The unit houses a 4 Litre bottle of Forever Wet inside.

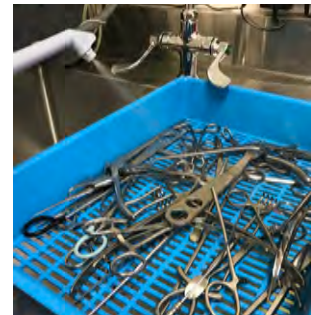

Wide spray pattern to cover all instruments evenly

Instructions For Use (IFU)

1. Check chemical level in bottle.
2. Turn power on.
3. Hold applicator wand 15cm – 20cm over instruments and squeeze lever.
4. Coat instruments thoroughly, moving the applicator side to side.

CAUTION:

Personal protective equipment should be worn when operating ForeverPrep™. Always point the ForeverPrep™ applicator wand away from you.