

SURGISTAIN®

Surgistain® removes water mineral staining, rust and corrosion from stainless steel surgical instruments safely and quickly within 15 minutes.

Surgistain® is a quick, safe, and efficient non-corrosive revitalizing solution that removes all water mineral staining, rust, pitting, and corrosion from stainless steel surgical instruments, trays, basins and trolleys. Used routinely as part of the Ruhof System, Surgistain® will help save money by maintaining the life and efficiency of your instruments, reducing repair and replacement costs. If used on a monthly basis in conjunction with our enzymatic cleaners, Surgistain® will restore the instrument's original finish and restore articulation by loosening box-joints. Prevents destructive pitting on ridges of micro and ophthalmic instruments

CAUTION

DO NOT USE ON CHROME OR SILVER PLATED INSTRUMENTS

ie: trach tubes, malleable ribbon retractors, uterine sound, malleable gallbladder scoops and dilators.

AVAILABLE IN

Cat No 345B – 4 x 4 Litres
Cat No 345B1 – 12 x 500ml



DIRECTIONS FOR USE

Stainless Steel Instruments

1. Mix one (1) part Surgistain with seven (7) parts hot water.
2. Place solution in a non-porous container.
3. Place clean instruments that are opened in the solution cautiously to prevent splashing for 15 minutes.

Gross or Heavily Stained Instruments Soak for 30 Minutes.

4. Remove instruments from Surgistain solution.
5. Rinse thoroughly.

Stainless Steel Trolleys and Trays

1. Mix one (1) part Surgistain with seven (7) parts hot water.
2. Using gloves, scrub thoroughly cleaned cart or tray with a hard bristle brush and hot solution.
3. Rinse with water.
4. Trolleys may require towel drying if manual trolley wash is used.

R **RUHOF** 
THE EXPERTS IN A BOTTLE

Peskett Solutions Ltd
Unit 4-5 Brampton Business Park
55 Brampton Road, Eastbourne, East Sussex, BN22 9AF
Tel: 01323 511038 Fax: 01323 503247
E Mail: info@ruhof.co.uk Web: www.ruhof.co.uk