


Endozime® AW Plus

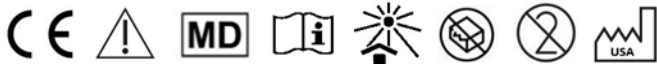

Use for washing all scopes (flexible, rigid), biopsy forceps, reusable surgical instruments including those made from metals, plastics, rubber, corrugated tubing, glass, and mirrors.

DIRECTIONS FOR USE:

1. Ensure the use by date on the label has not expired.
2. Follow the appropriate medical device manufacturer IFU to ensure all cleaning instructions are followed, such as submerging instruments fully, thoroughly cleaning and rinsing lumen and brushing off any stubborn soil.
3. **Manual Cleaning** - Add the Endozime® AW Plus to the water in the ratio 0.5 oz to 1 gallon of Water (4ml to 1 Litre of water). To reduce mineral deposits being left on a medical device, it is recommended that the rinse stage is carried out using tap, sterile, or demineralised water.
4. **Automatic Cleaning Devices** - Use detergent reservoir or use the dosing system following the machine manufacturer's IFU. Expose the medical devices to the cleaning solution for a minimum 2 minutes. To reduce mineral deposits being left on a medical device, it is recommended that the final rinse stage is carried out using R.O. water.
5. Ensure the cap is placed back on the bottle when not in use.

WARNINGS: See SDS (EU_ENG SDS-34514) for full details. In the event of accidental exposure, rinse exposed area thoroughly in water. Do not swallow. Do NOT induce vomiting if ingested, drink plenty of water. Avoid splashing concentration in eyes and avoid prolonged contact with skin. If symptoms persist call a healthcare professional.



www.ruhof.com



Advena Ltd. Tower Business Centre, 2nd Floor,
Tower Street, Swatar, BKR 4013, Malta



XXXXXXXXXXXX



Peskett Solutions Ltd.
Unit 4, Brampton Business Park,



XXXXXXXXXXXXXXXX



55 Brampton Road,
Eastbourne, East Sussex,
BN22 9AF. United Kingdom.



YYYY/MM/DD

+44 (0) 1323 511038



34514-XX

Scannable code QR or
Bar

LB-729 rev.0 012623